

Rubo Medical Imaging

DICOM Viewer 2.0

- ...including
- DICOM Communication
 - DICOM Anonymizer
 - DICOM Parser
 - DICOM DVD Burner
 - Waveform viewer
 - IVUS Longitudinal viewer

Single license installation instructions, USB Key

The setup file of the 'Rubo DICOM Viewer 2.0' will install the software and drivers automatically. If you encounter issues, please read below.

Download and install the DICOM Viewer

1. Download the setup file of the DICOM Viewer here:
http://www.rubomedical.com/dicom_viewer. Administrator privileges are required.
2. Read and accept the end user license agreement and click through all the setup screens.
3. The DICOM Viewer is installed, the Gemalto/Sentinel/HASP drivers are installed, the USB hardware key is plugged in and the light in the key is switched on: press 'OK' to finish and start the DICOM Viewer.

Install Gemalto/Sentinel/HASP (USB key) Run-time environment manually

If you encounter problems installing the Gemalto/Sentinel/HASP environment, please follow these steps:

1. Download the driver setup file here: www.rubomedical.com/sentinel. Choose between command-line or GUI installation. Administrator privileges are required to install.
2. Click through all setup screens.
3. When the installation has completed, press 'Finish' and plug in the USB hardware key and wait for the message 'Your device is ready to use'. This may take a few minutes. You can check the progress by clicking on the 'Device Driver Installation' tray icon.
4. The USB hardware key is plugged in and the light in the key is switched on, you can now start the DICOM Viewer.